

FOCUS 40 Blue - Adjusting Braille Settings

Introduction

The Focus 40 is a refreshable Braille display that allows users to use the built-in Perkins style keyboard and additional keys to fully access computer functions when paired with JAWS.

This tutorial assumes

- That you have a copy of JAWS installed on your computer
- That the FOCUS 40 is connected to the computer via Bluetooth or USB

Learning Objectives

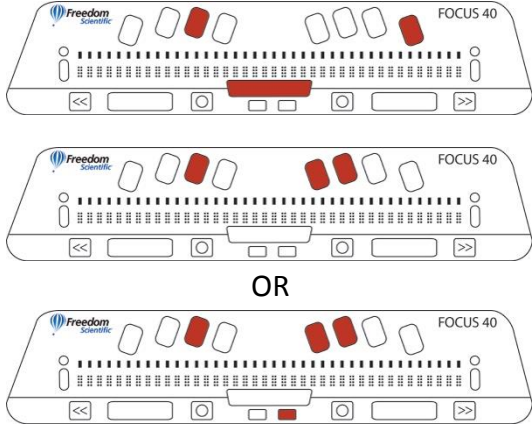
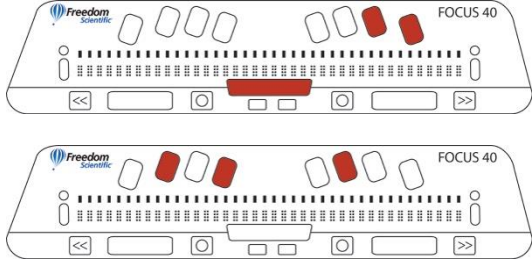
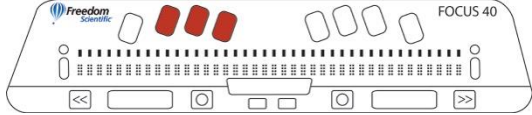
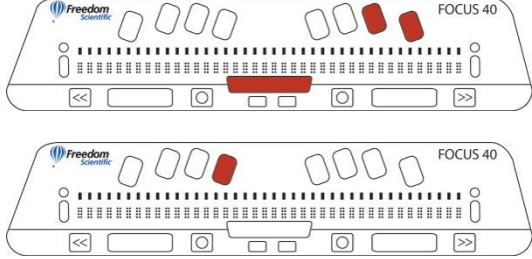
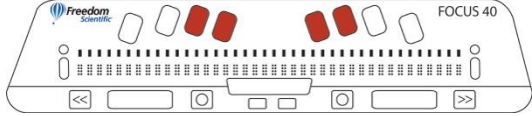
Completion of this tutorial will give you experience with the following:

- Adjust the Braille Settings for user's preferences.

Case Study

Annie is a student who uses Braille to access her curriculum. She uses a computer with a FOCUS 40 refreshable Braille display to access her computer.

Adjusting Dot Firmness on FOCUS 40 Blue

<p>1</p>	<p>Launch JAWS by pressing INSERT+J. Dot 2-8 CHORD, J</p> <p>Or</p> <p>RIGHT SHIFT+J</p>	 <p>OR</p>
<p>2</p>	<p>Press ALT+O to access the options menu Dot 6-8 CHORD, O</p>	
<p>3</p>	<p>Press L to select the Braille menu.</p>	
<p>4</p>	<p>Press ALT+A to select Advanced menu. Dot 6-8 CHORD, A</p>	
<p>5</p>	<p>Press G to go to General submenu item.</p>	

SETBC

6	Press F6 to move the cursor to the Advanced pane. Dot 1-8 CHORD, F	
7	Interact with the General Settings pane by using TAB or SHIFT-TAB to move around the settings.	
8	Use the ROCKER KEYS to interact with combo boxes.	
9	Press SPACEBAR to check or uncheck checkboxes.	
10	Press ENTER to accept the settings and exit JAWS Settings Center.	
11	Press ALT+O to accept the settings and exit the Options menu.	