

GoTalk 9+

Introduction

GoTalk 9+ is an easy to use communication device for all ages. Simple to record, built-in overlay storage, and both record and level locks.

- Records up to 45 messages on 5 levels, plus 3 'core' messages that stay the same on each level
- GoTalk9 supports students of all ages in school, home, or social situations

Integration Ideas

- Initiate conversations
- Support daily activities
- Participate in group instruction
- Lead the class in games and activities



How to Program the GoTalk 9+

- Make an overlay and slide it in (make one for each level and store)
- Press and release the record button. The level button will show green, the record light will show red. Press the key you want to record to, and speak to record your message. (The red record light will blink)
- When done, press any key to stop recording.
- Record one overlay at a time. Change the level, insert a new overlay, and record all levels in one sitting.

How can I learn how to use it?

Here is a resource from the manufacturer that might help you learn how to use the GoTalk9+

- <https://www.youtube.com/watch?v=IloAfacbvi0&feature=youtu.be>

Here is a video from a communication therapist, explaining how to use the GoTalk9+

- <https://youtu.be/IENNufErOLA>